# DEAR PATIENT,


the instructions will help you to prepare the MENOGON® HP pre-filled pen properly so that you can then inject the medicine optimally.

#### Please note the following information:

- The MENOGON® HP pre-filled pen must not be used until your doctor has shown you how to prepare and inject the pen correctly.
- Read the instructions for use in the pen package completely before using the MENOGON® HP pre-filled pen and also each time you get a new pen.
- Use a new needle for each injection.


Contact your doctor or pharmacist if you have any questions about the use of the MENOGON® HP pre-filled pen.



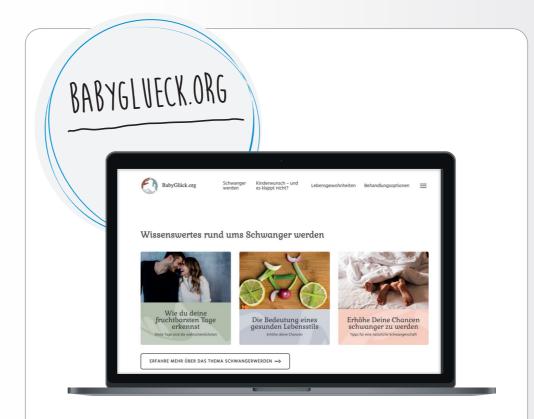


### The great knowledge portal for your desire to have children

Comprehensive information and tips about your desire to have a child and treatment options at the Fertility Centre.

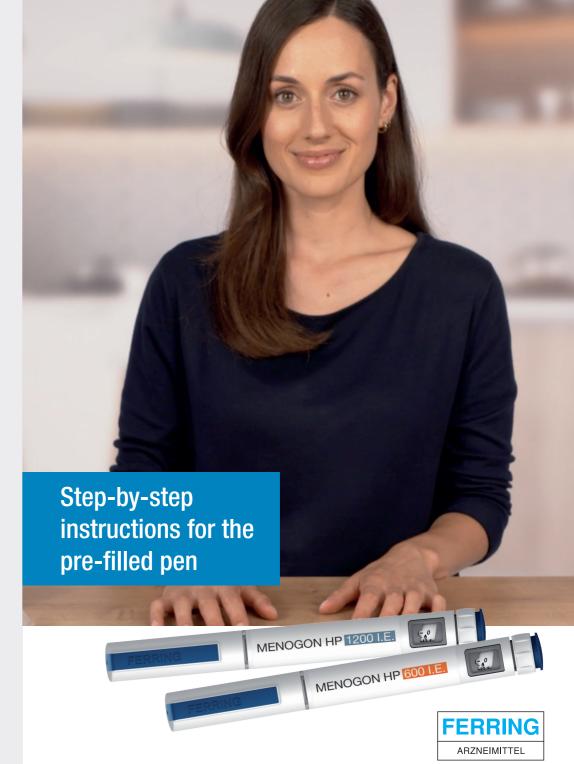



### COMPONENTS

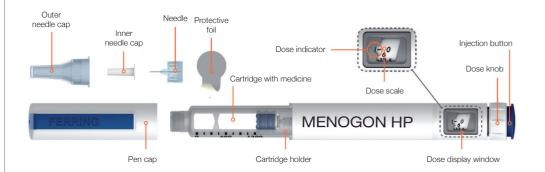

The MENOGON® HP pre-filled pen is available in two different dosage sizes.



## BEFORE USE

- Wash your hands.
- Check if the pen is damaged. Do not use the pen if it is damaged.
- Check that the medicine in the cartridge looks clear and does not contain particles. Do not use the pen if the medicine in the cartridge does not look clear or contains particles.
- Make sure you have the correct pen with the correct dosage size.
- Check the expiration on the pen label.

### HOW DO I PREPARE THE INJECTION?



#### 1. Attaching the needle

Pull off the pen cap. Remove the protective foil from the needle and attach the needle. Remove the outer and inner needle caps.

Do not throw away the outer needle cap!



#### 2. Priming

Before using the pen for the first time, you must deflate it. If the pen has already been used, proceed directly to step 3.

Turn the dose knob clockwise until a droplet symbol appears in dose display window **d** and lines up with the dose indicator.



Hold the pen with the needle pointing upwards. Tap the cartridge holder with your finger so that all the air bubbles in the cartridge rise to the top.



Continue to hold the needle upwards (away from the face) and press the injection button down as far as it will go until the number «0» is displayed in the dose display window and a drop of liquid comes out of the tip of the needle. If no drop comes out, repeat Step 2 until a drop appears.

## HOW DO I PERFORM THE INJECTION?



#### 3. Dose selection and dose correction

Turn the dose knob clockwise until the prescribed dose lines up with the dose indicator in the dose display window. The space between two numbers is divided by one dash, each dash corresponding to a dose increment of 6.25 I.U. The dose can be corrected up or down without loss of medicine by turning the dose knob in one direction or the other until the correct dose appears in the dose display window.



#### 4. Injection of the prescribed dose

Clean the injection site with an alcohol swab. Squeeze the skin and insert the needle straight as your doctor has shown you. The dose display window should be visible to you during the injection. Press the injection button down to the stop and hold it there until the number «0» appears in the dose display window.

Wait 8 seconds, then you can release the injection button and slowly remove the needle from the injection site.



#### 5. Removing the needle

Carefully push the outer needle cap onto the needle. Unscrew the needle from the pen by turning it anticlockwise. Dispose of the used needle. Place the pen cap firmly on the pen to protect it between injections.

#### Dividing the dos

If you cannot adjust the prescribed dose, there is not enough medicine left in the pen. You will need a new pen to complete the dose. Please proceed as follows: Note how much medicine is left in the pen and calculate the remaining dose. First inject the medicine remaining in the pen and then inject the calculated additional dose with the new pen (step 1 – 5).